

BinaxNOW COVID-19

Home Test Kit Instructions



Requirements

- a) Windows <u>or</u> Mac/Apple laptop with a webcam (or a desktop computer with an adjustable/moveable webcam), microphone, and speaker
- b) An Apple phone/tablet <u>or</u> an Android phone/tablet
- c) An email address
- d) An internet connection

Before Picking up a Test Kit

1. Download the NAVICA app from the <u>Apple App Store</u> or the <u>Google Play App Store</u> from your smartphone/tablet



- 2. Open the NAVICA app on your smartphone/tablet and Create a NAVICA account with a <u>username and password</u>
- 3. If the person being tested is age 17 or younger), then tap the TEST SITE icon at the bottom of the NAVICA app and then tap the profile icon at the upper right of the screen to add a MANAGED PROFILE for the minor to be tested







When Picking up the Test Kit or When Delivered

- 1. Provide your name, date of birth and email address to the person handing you a test kit
- 2. Show the person you have download the NAVICA App
- 3. Take the test kit home

Time to Start the Test

- 1. On a computer or laptop go to <u>ohio.emed.com</u>
- 2. Click the START TESTING button on the page



3. Click the LOGIN WITH NAVICA button on the page and use your NAVICA <u>username and</u> <u>password</u>







4. Answer the series of questions to prepare for the visit with the eMed proctor and click CONTINUE. This will begin the testing session with the eMed proctor

Do you i Certified	neet the minimum technical requirements above for a video visit with eMed I Guide?
O Yes	
⊖ No	
Are you	free for the next 20 minutes?
O Yes	
⊖ No	
Do you l	nave your unopened test available?
O Yes	
O No	
Are you	hands washed, sanitized and dry?
O Yes	
⊖ No	
ls this yo	ur first COVID-19 test?
O Yes	
⊖ No	
Do vou l	ive in a lona-term care facility, nursina home, or homeless shelter?

- 5. Provide your name and date of birth to the eMed proctor
- 6. Follow all directions from the eMed proctor to take the test

Results

1. After you complete the test you will receive a "pass" on your NAVICA smartphone/tablet app that contains your test results. Tap PASS on the NAVICA app to view your results and pass







- 2. If you test negative, you can show your pass to your school or employer from your smartphone/tablet
- 3. Results could also be found 60 minutes later by using the results tab in the app

5:39				⊽ ■
C Back				
Resu	lts			
	CO	VID-19 to	est resu	lt
is negat	ive.			
A negative the virus t in your sa	e test result hat causes mple at the	means tha COVID-19 time of yo	at protein: were not ur test.	s from found
	BR St	nare Result	1	
COVID-1	9 Test Del	tails		
Results F Mar 29,	Reported On 2021, 11:23	AM EDT		
Name				
Date of E	Birth			
Test Res	ult			
Abbott T	e est DW	1		
BinaxNO				
BinaxN0 Test Typ Antigen	e			4
BinaxNC Test Typ Antigen	e			
BinaxNO Test Type Antigen Lot	e 	E.	F	0



